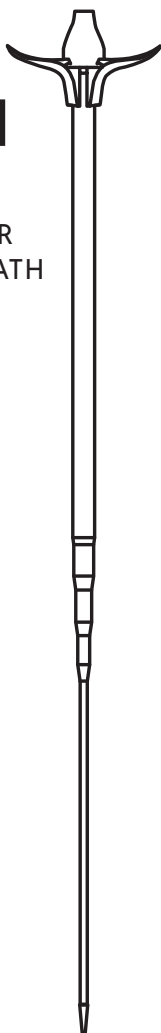
AVANOS*

ENTERAL ACCESS DILATION SYSTEM

TELESCOPING DILATOR
WITH PEEL-AWAY SHEATH

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AVANOS*

Enteral Access Dilation System

Telescoping Dilator with Peel-Away Sheath

Instructions for Use

Rx Only: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Intended Use

The AVANOS* Enteral Access Dilation System is intended to facilitate stoma tract dilation prior to placement of the AVANOS* MIC* and MIC-KEY* brand Enteral Feeding Tubes.

It is recommended that this dilator be used only with the AVANOS* MIC* and MIC-KEY* brand of Enteral Feeding Tubes.

Contraindications

Contraindications include, but are not limited to ascites, colonic interposition, portal hypertension, gastric varices, peritonitis, aspiration pneumonia and morbid obesity (stoma lengths longer than 10 cm).

Do not use for vascular access.

Note: Verify package integrity prior to opening. Do not use if package is damaged or sterile barrier compromised.

⚠ Warning: Do not reuse, reprocess, or resterilize this medical device. Reuse, reprocessing, or resterilization may 1) adversely affect the known biocompatibility characteristics of the device, 2) compromise the structural integrity of the device, 3) lead to the device not performing as intended, or 4) create a risk of contamination and cause the transmission of infectious diseases resulting in patient injury, illness, or death.

Dilation

⚠ Caution: Prior to dilation, ensure that the stomach is in apposition to the anterior abdominal wall. Excess lubricant may cause difficulty in gripping the dilator segments.

1. Remove the dilator from the package.
2. Verify functionality prior to procedure by advancing each dilator sleeve distally until the physical stop is felt. Once verified retract all sleeves to their original position.
3. Verify that the skin incision is adequate to accommodate the peel-away sheath.
4. Apply water soluble lubricant to the stoma site.
5. Advance the dilator tip over the pre-positioned guidewire, through the tissue and into the stomach using a clockwise / counter clockwise twisting motion. **(Fig 1)**
6. Advance the next dilator sleeve into the stomach until a physical stop is felt.
7. If placing a MIC-KEY* Low-Profile Feeding Tube, continue dilation through the red sleeve. Then remove the dilator from the stoma and measure the stoma length with a MIC-KEY* Stoma Measuring Device (Sold Separately). After measuring remove the Stoma Measuring Device and resume dilation by advancing the dilator over the guidewire, through the stoma tract and into the stomach.
8. Continue dilation until all dilator sleeves have been advanced.
9. Twist the dilator hub to release the peel-away sheath from the dilator. **(Fig 2)**
10. Lubricate the exterior of the peel-away sheath with a water soluble lubricant and advance the sheath through the tract and into the stomach.
11. Remove the dilator, leaving the guidewire and the peel-away sheath in the stomach with the remainder securely maintaining position through the tract and exiting the stoma site.
12. Select the appropriate AVANOS* MIC*, MIC-KEY* Enteral Feeding Tube while maintaining stomach and stoma tract access via the pre-positioned peel-away sheath.
13. Advance the tube down the peel-away sheath and into the stomach.
14. After the tube has been advanced through the peel-away sheath and is in position in the stomach, peel the sheath away from the tube, remove and dispose of according to facility protocol. **(Fig 3)**
15. Complete the placement procedure according to the AVANOS* MIC* and MIC-KEY* brand Enteral Feeding Tube Instructions for Use.
16. Confirm that the enteral feeding tube is positioned in the GI tract prior to feeding.

Not for Vascular Use	 Single Use Only	 Sterilized Using Ethylene Oxide
 Do not use if package is damaged	 Do not resterilize	Not made with natural rubber latex
Rx Only	 Caution	 Consult instructions for use

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[illegible]

www.avanos.com

Distributed in the USA by Avanos Medical Sales, LLC,
5405 Windward Parkway, Alpharetta, GA 30004 USA.
In USA, 1-844-4AVANOS (1-844-428-2667). • www.avanos.com

 Avanos Medical, Inc., 5405 Windward Parkway,
Alpharetta, GA 30004 USA.

 Avanos Medical Belgium BVBA,
Leonardo Da Vincilaan 1, 1930 Zaventem, Belgium.
Sponsored in Australia by Avanos Medical Australia Pty Ltd,
475 Victoria Avenue, Chatswood, NSW 2067, Australia.

*Registered Trademark or Trademark of Avanos Medical, Inc.,
or its affiliates. © 2018 AVNS. All rights reserved. 2019-01-28
15-M1-193-01 / 70210622